

Welcome to your new Bari, Bariatric conversion suit for the 13XX series Rescue Randy.

The principle behind this product is to provide you with a simulator to challenge your skills in the extraction of bariatric patients. The fact that you can build your patient in the situation allows for easy scenario set up where two people can comfortably place the manikin then present a problem for 10 people to resolve.

It is possible to start your training using a 1338 25lb manikin with the suit full of air to solve problems of bulk without the weight.

When the suit is full and donned on a 1345 165lb manikin the total weight is approximately 400lb!

It is imperative that all users of this product perform adequate risk assessment for the exercise they are about to perform. Part of this assessment should consider the possibility of water damage should the suit be damaged in the course of the exercise, the suit holds approximately 185 litres of water at capacity. Simulaids Ltd. will not accept any responsibility for damage or injury as a result of using this product.

To don the suit, start with the legs, pull the suit on to waist level, then place the manikin in a seated position on the floor and place the arms into the suit. Bring the arms fully into the suit to shoulder level. You can then simply fasten the Velcro up the back of the manikin and you are ready to fill. Only use fresh (tap) water.

Do not attempt to lift the manikin by using the opening flap at the back of the manikin, this can cause of tear to start.

There is a soft collar to finish the "fat" effect

There are three openings to the suit, depending on your scenario you can fill from any. Whilst filling leave one valve, distant from the fill point, open to allow for air escape. To get optimum filling, it is best to have the manikin on it's back.

When draining the suit, make sure you are in an environment that can take a large volume of water, preferably near a drain, open all three valves. Position the manikin so gravity assists the draining. Empty as much water from the manikin as possible to allow easier movement when removing the suit from the manikin. If possible inflate the suit to facilitate the final draining of residual water. Hang the suit to dry after use.

It is safe to use a dilute sterilising solution periodically to kill any microbial activity in the suit.

This product has been constructed with marine grade trilaminate material to give the optimum durability and service.

Notwithstanding it must be noted that you should handle the product as you would a patient, if you drop your patient during extrication you may well kill them, you may do the same to your new suit! If you catch your patient on sharp corners you will damage them, your new suit may be equally harmed!

Simulaids Ltd warranty the manufacture of the suit for a period of 12 months, this does not include wear and tear through use.