

# ASSEMBLY INSTRUCTIONS

## (USE AND CARE SHEET)



## TRAINING MANIKIN

THANK YOU FOR INVESTING IN CPR PROMPT PROFESSIONAL TRAINING PRODUCTS

If you have any questions about your manikins  
or need to order extra lung bags:

Please call us **TOLL FREE** at 1-800-558-9595

Write to:

Nasco

901 Janesville Ave.

Ft. Atkinson, WI 53538-0901

E-mail to: [lifeform@eNasco.com](mailto:lifeform@eNasco.com)

Website: [www.cprprompt.com](http://www.cprprompt.com)


## TO START...

- Use this sheet for instructions on how to assemble your manikins.
- Clean the manikin heads before you start (refer to cleaning instructions at the end of this sheet).
- Use only “foam manikin” AED training pads (refer to part numbers at the end of this sheet).
- If you have any problems with assembling these manikins, refer to the troubleshooting guide located at the end of this sheet (or call us toll free at 1-800-558-9595).


## CONTENTS & PARTS

Each manikin comes with a supply of face shield lung bags and an insertion tool.

### ADULT/CHILD MANIKIN

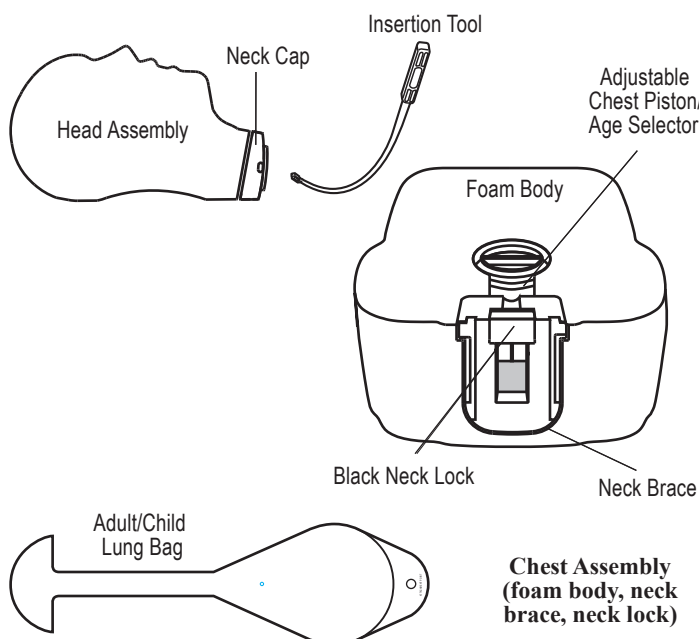

### INFANT MANIKIN


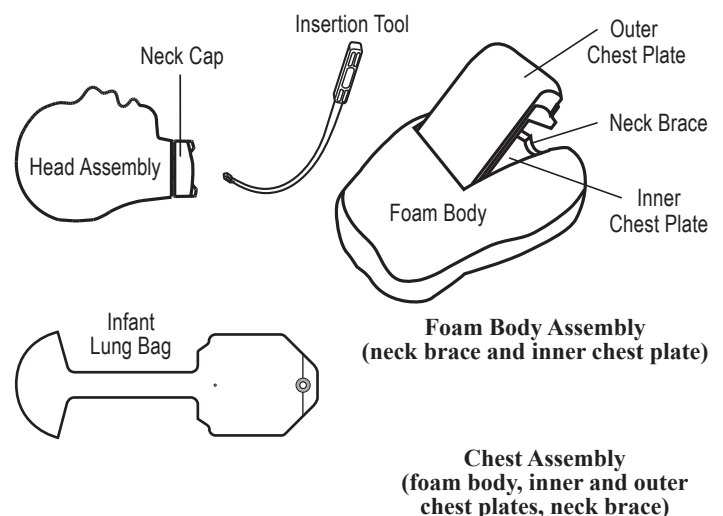


#### TIP:

**Before installing lung bag** – Wrinkle the neck and pouch of the bag by rolling it between your thumb and fingers. This will help ensure that the air flows freely into the lung bag during use.

**10**


- a) Turn the age selector dial on the chest piston so the pin is next to the ADULT or CHILD arrow (whichever age you are learning).
- b) Replace the chest piston in the chest cavity so that the age selector dial is visible in the body.

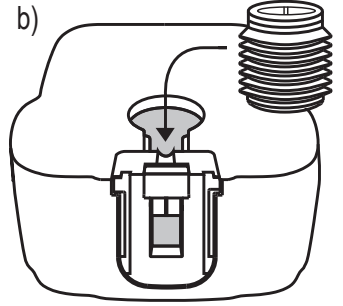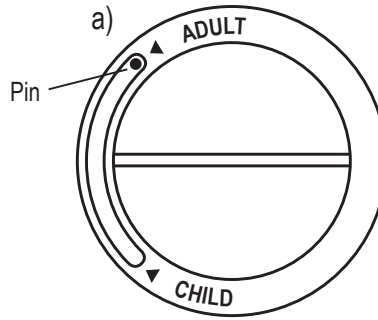


## INFANT ASSEMBLY

**1**


- Feed the insertion tool through the neck and up so hook extends out of the mouth.

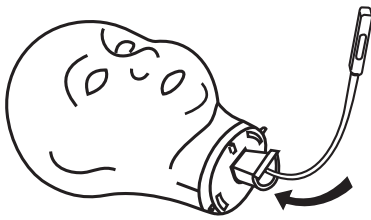**2**


- Place bag on face of manikin.
- Attach hook of tool to bottom hole on bag.

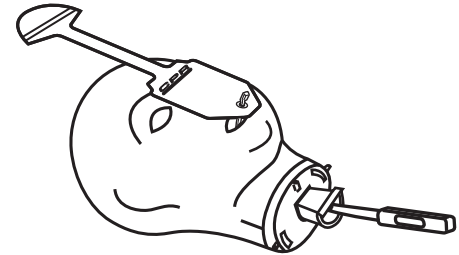**3**


- Gently pull insertion tool until the bag extends out of the neck opening. Face shield portion should cover mouth.

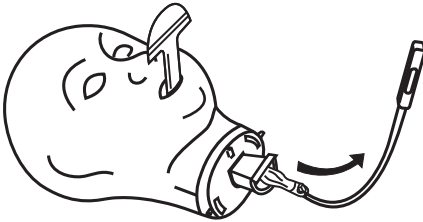**4**


- Remove adhesive strip and fold tabs over mouth.
- Attach head to torso.

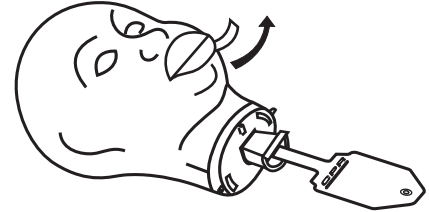**5**


- Open the infant chest.
- Grip the collar of the outer chestplate and lift up.

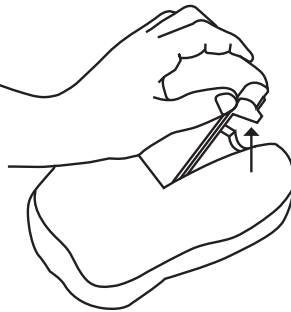**6**


- Slide the head and lung bag into the neck brace. Head facing up
- With the head facing up, line up the two pins on the head with the slots in the neck brace.
- Guide the pins into the slots and let the head drop down. Front of body

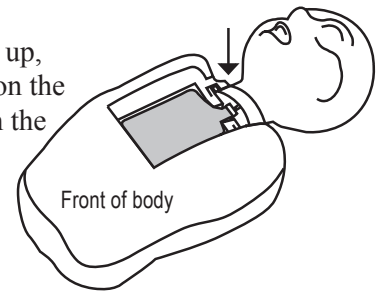**7**

- Center the lung bag on top of the inner chest plate.
- Make sure that the bag is laying flat, and there are no "kinks."

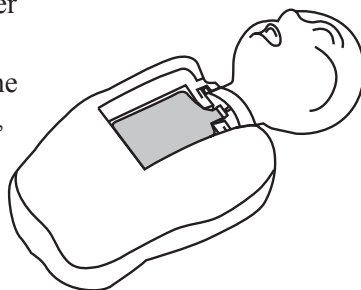**8**


- Replace the outer chest plate.

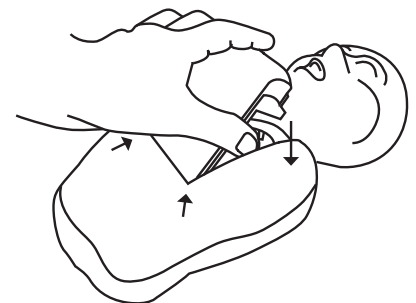

## ADULT/CHILD ASSEMBLY


1

- Feed insertion tool through the neck and up so hook extends out of the mouth.

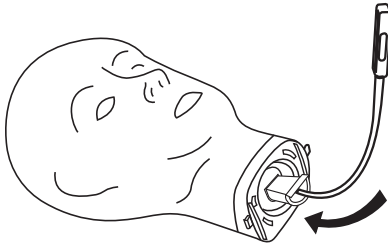


2

- Place bag on face of manikin.
- Attach hook of tool to bottom hole on bag.

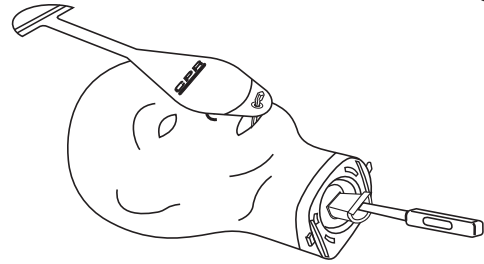


3

- Gently pull insertion tool until the bag extends out of the neck opening. Face shield portion should cover mouth.

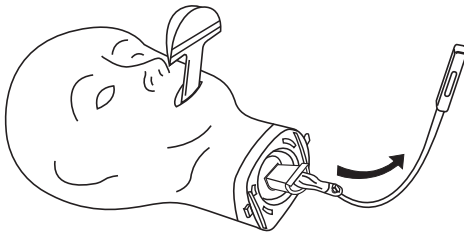


4

- Remove adhesive strip and fold tabs over mouth.
- Attach head to torso.

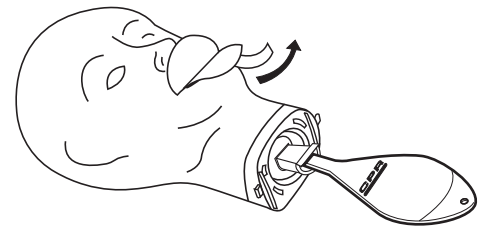


5

- Pull the piston out of the bottom of the chest cavity.

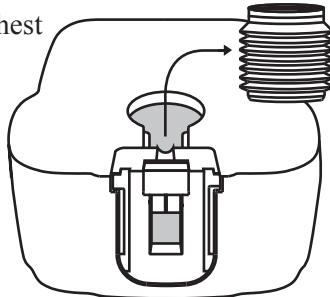


6

- Remove the black neck lock from the neck brace.
- Squeeze the top of the black neck lock and pull up and out of the neck brace.

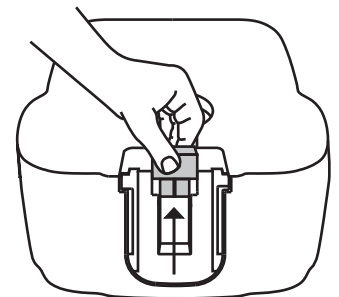


7

- While holding the head, pull the lung bag through the opening in the neck brace and lay the bag flat, centered in the chest cavity.

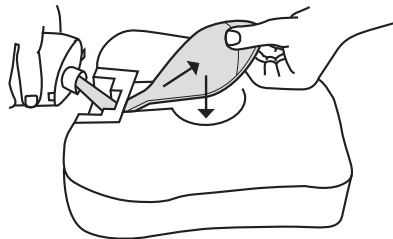


8

- With the head facing down, line up the two pins on the head with the 2 slots in the neck brace. Push the pins until they slide into the slots and the head drops down into place.

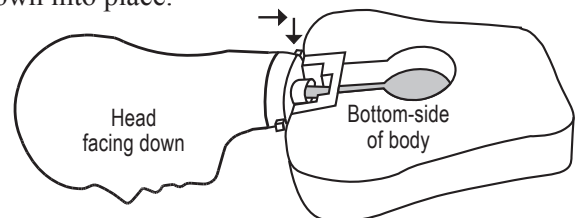

9

- Replace the black neck lock in the neck brace to hold the head in place.

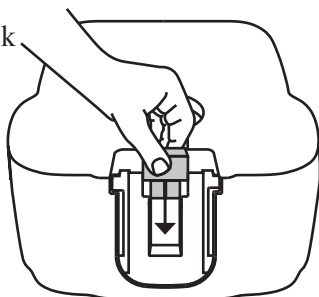

***Make sure that there are no kinks in the lung bag.***

## TROUBLESHOOTING GUIDE

Some suggestions if you encounter any of these problems...

***If you cannot get the chest to rise when you blow... make sure that:***

- The head is tilted back.
- The nostrils are covered.
- The lung bag is properly positioned: it is not twisted...it is laying flat inside the chest cavity
- The lung bag is properly attached to the head.
- The bag has been fed through the Neck Cap.
- The bag has not been torn during assembly
- The Neck Cap is in the correct position (“FRONT” of neck cap should be lined up with the front of the head).

***If you do not hear a click when you compress the chest of the Adult/Child manikin:***


- Make sure that you are pressing deep enough (you should hear the click when you compress the foam about one third of its depth).
- Check the Adjustable Chest Compression Piston and make sure that the black pin in the Age Selector Dial is pushed all the way over to the Adult or the Child – depending on the age you are learning.

## CLEANING INSTRUCTIONS

### NECK CAP

**Before cleaning heads, remove neck cap:**

- To remove the neck cap, twist the neck cap counter clockwise like the lid on a jar (about a quarter of a turn).

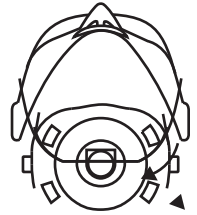

### HEADS

**Heads can be cleaned by the following methods:**

- Run them through a dishwasher, OR...
- Wash heads with soap and water. Then soak 10 minutes in a solution of 1/4 cup bleach mixed with one gallon of water, OR...
- Sanitize with alcohol wipes.
- After cleaning head, replace the neck cap by twisting the neck cap back onto the head. Make sure the “FRONT” of the neck cap is lined up with the front of the head.

### BODIES

- The bodies can be safely cleaned with a bleach/water solution (see details under “HEADS” section); alcohol, or any other sanitizing agents.\*

\*Manikin bodies are foam based and are not waterproof. They should not be submerged in water.

## AED TRAINING

- Use only “foam manikin” AED training pads:
  - LF06501U Medtronic Physio-Control Style
  - LF06502U Heartstream Style
  - LF06503U Survivalink Style
- Peel pads off gently after each training session, being careful not to rip the skin coating.

## OTHER AVAILABLE CPR PROMPT® PRODUCTS

**LF06928U** CPR Prompt® Small Blue Bag (17" x 8 1/2" x 11 1/4")

**LF06929U** CPR Prompt® Large Blue Bag (30" x 13" x 19")

**LF06940U** CPR Prompt® Small Gray Bag (17" x 8 1/2" x 11 1/4")

**LF06941U** CPR Prompt® Large Gray Bag (30" x 13" x 19")

**LF06200U** CPR Prompt® 200 Audio Rescue & Practice Aid  
Key Chain

**LF06204U** CPR Prompt® 300 Audio Rescue & Practice Aid

**LF06202U** CPR Prompt® CPR Responder Kit

**LF06943U** Insertion Tool — 5-Pack, Adult/Child/Infant

**LF06923U** Black Neck Lock

**LF06206U** Adult/Child Face Shield/Lung Bags –10-Pack

**LF06208U** Adult/Child Face Shield/Lung Bags – 100-Pack

**LF06207U** Infant Face Shield/Lung Bags – 10-Pack

**LF06209U** Infant Face Shield/Lung Bags – 100-Pack

## CPR BLUE AND TAN ADULT/CHILD MANKIN PART NUMBERS

**LF06911U** Blue Head Assembly

**LF06918U** Blue Neck Cap

**LF06913U** Blue Adjustable Chest Piston/Age Selector

**LF06932U** Blue Chest Assembly (foam body, neck brace, neck lock)

**LF06910U** Tan Head Assembly

**LF06919U** Tan Neck Cap

**LF06912U** Tan Adjustable Chest Piston/Age Selector

**LF06931U** Tan Chest Assembly (foam body, neck brace, neck lock)

## CPR BLUE AND TAN INFANT MANKIN PART NUMBERS

**LF06915U** Blue Head Assembly

**LF06926U** Blue Neck Cap

**LF06922U** Blue Outer Chest Plate

**LF06925U** Blue Foam Body Assembly (neck brace & inner chest plate)

**LF06935U** Blue Chest Assembly (foam body, inner & outer chest plates, neck brace)

**LF06914U** Tan Head Assembly

**LF06927U** Tan Neck Cap

**LF06920U** Tan Outer Chest Plate

**LF06924U** Tan Foam Body Assembly (neck brace & inner chest plate)

**LF06936U** Tan Chest Assembly (foam body, inner & outer chest plates, neck brace)