

# Export Brad Mouthpiece



Instructions for cleaning and disinfecting the Export Brad Mouthpiece, item 2023EXPORT (10 pack).

## Clean & Disinfect

- Steps:**
1. Clean as usual with your local C.D.C. Protocol.
  2. To remove from solution:
    - a. Hold mouthpiece with the back of it towards the palm of your hand.
    - b. Squeeze the cheeks.



- c. Pour cleaning solution from the hole in the mouth.
  - d. Continue squeezing the mouthpiece and shake it vigorously 2 or 3 times to assist in removing excess moisture
3. Place mouthpiece onto its cheek on a flat surface.
  4. Let air dry 1 hour to ensure that the interior of the mouthpiece thoroughly dries.