



PO Box 1289 - 16 Simulaids Dr - Saugerties NY 12477

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com

RESCUE RANDY ADULT RESCUE MANIKIN



FIREFIGHTER COMBAT
CHALLENGE



RESCUE RANDY ADULT RESCUE MANIKIN

This manual will familiarize the user with the function, care and maintenance of the RESCUE RANDY rescue training manikin. As with all SIMULAIDS' products, RESCUE RANDY is made to exacting standards and will provide trouble-free service when the few guidelines and simple steps of care and maintenance are followed.

FEATURES

Randy closely replicates the weight, feel and resiliency of a human body. Randy is articulated and can be used in various life-saving exercises including auto extrication, application of anti-shock trousers and other emergency evacuation procedures. **NOTE: UNDER NO CIRCUMSTANCES SHOULD THE EYE HOOK ON THE BACK OF THE MANIKIN BE USED TO SUSPEND THE MANIKIN FROM ANY HEIGHT.** Should the exercise call for suspending the manikin, Randy should be placed in a harness specifically designed for such procedures.

WATER RETRIEVAL EXERCISES use a manikin specifically designed to mimic a drowning victim such as Simulaids' **WATER RESCUE MANIKIN.**

CONSTRUCTION

Randy's body is suspended on a skeleton of steel and flexible cable. Some enlargement of the body segments (joints), cable connector openings or gaps between the body joints may develop with extended use without affecting the serviceability of the manikin.

CARE AND MAINTENANCE


Since Randy takes the place of a live victim in your training program, it should be maintained with care and treated as you would any other valuable rescue training tool. It will withstand rugged use. However, it is recommended that it not be dropped nor subjected to temperatures above 100°F (except for short periods of time). These precautions will prevent deformation of the body and possible damage to the internal skeletal structure.

Care should be taken so that the body and limbs are not punctured or torn. Randy features articulated limbs. However, care should be taken not to twist the arms and legs into positions that a human body would not normally withstand.

ATTENTION: DO NOT BIND EXTREMITIES OR EXTREMITY JOINTS, either together or separately, in an effort to control joint motion during dragging and moving exercises.

After each training exercise, or when needed, Randy should be washed with hot, soapy water (do not immerse in water). The use of solvents or strong cleansers is not recommended. However, an "all-purpose" cleaner (i.e. 409 or Fantastic) may be used. Randy may be used in low temperatures, but, if so, you can expect the vinyl body to become stiff. It will return to normal flexibility once it is returned to a higher temperature.

RANDY LEG ASSEMBLY DIAGRAM

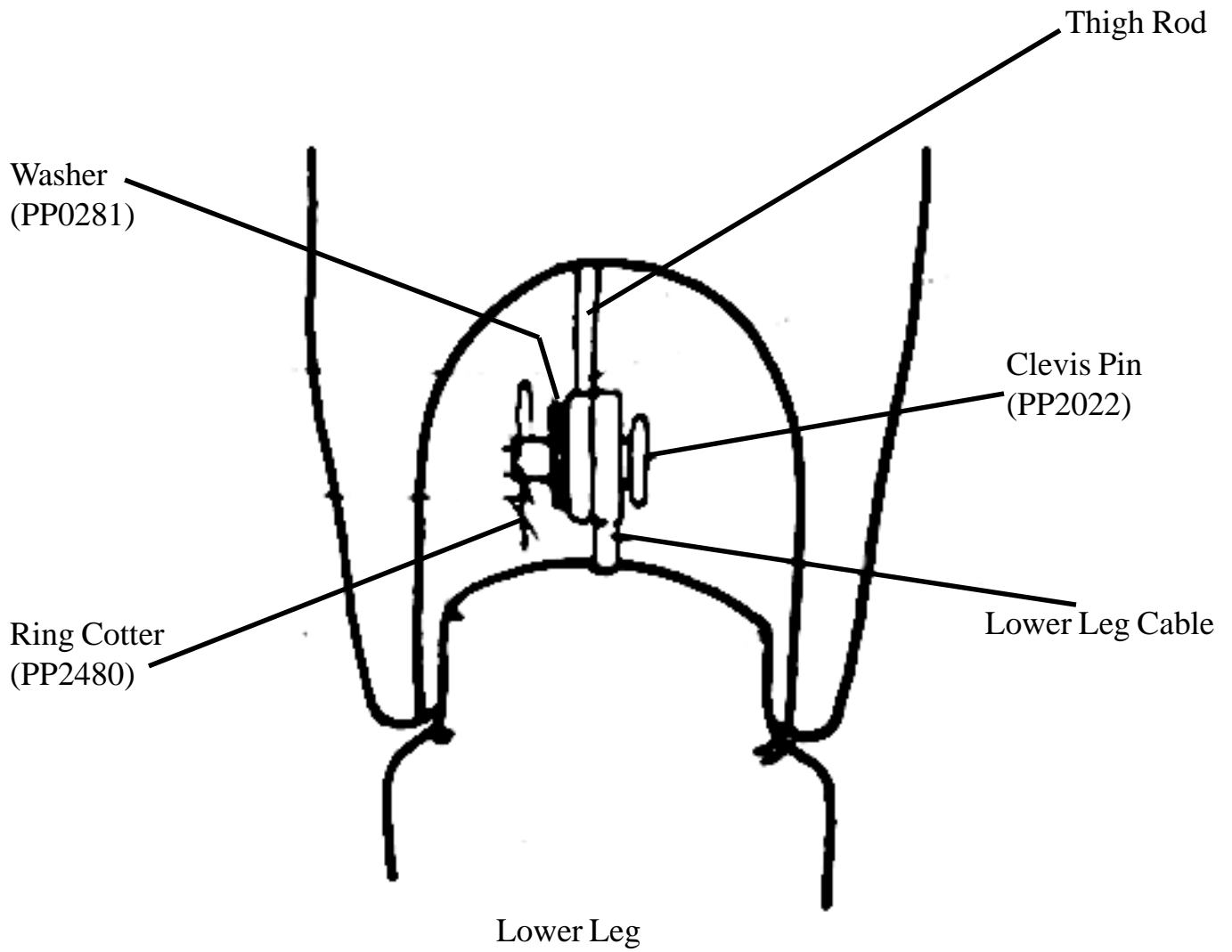

STORAGE

If possible, Randy should be stored at normal room temperature (62°F to 80°F). The optional Carry/Storage Bag is an excellent way of keeping your manikin clean and damage-free while not in use.

RESCUE RANDY

| | |
|--|----------|
| Model No. 1338 | 55 lbs. |
| Model No. 1335 | 105 lbs. |
| Model No. 1344 | 145 lbs. |
| Model No. 1345 | 165 lbs. |
| Model No. 1434 Rescue Randy Combat Challenge Manikin | 145 lbs. |
| Model No. 1435 Rescue Randy Combat Challenge Manikin | 165 lbs. |
| Model No. 1475 IAFF Rescue Randy | 165 lbs. |

ACCESSORIES

Weighted Vests

| | |
|---|---------|
| Model No. 1112 (<i>Small, Medium, Large, X-Large</i>) | 20 lbs. |
| Model No. 1114 (<i>Small, Medium, Large, X-Large</i>) | 30 lbs. |
| Model No. 1115 (<i>Small, Medium, Large, X-Large</i>) | 40 lbs. |
| Model No. 1118 Small IAFF | 50 lbs. |
| Model No. 1120 Medium IAFF | 50 lbs. |
| Model No. 1125 Large IAFF | 50 lbs. |
| Shoulder weights Model 1000 12 1/2 lbs./pr | 25 lbs. |

Carry/Storage bags

- Model No. 1373 Full Body Adult Manikin
- Model No PP2094 Combat Challenge Mankin Carry Bag
- Model No. 960 Soft Stretcher (21" x 35")

Return Policy

Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

Limited Three (3) Year Warranty

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.

PO Box 1289 - 16 Simulaids Dr - Saugerties NY 12477
 (845)-679-2475
 Toll Free: (800)-431-4310 Fax: (845)-679-8996
 www.simulaids.com